

- [Syphilis TP](#)
- [Features & Benefits](#)
- [Performance Data](#)
- [Determine Syphilis TP How to Use](#)
- [Determine Syphilis Product](#)
- [Specifications](#)
- [Determine Syphilis FAQ](#)

## Performance Data

### Determine Syphilis TP



**Determine® Syphilis TP** delivers outstanding performance versus industry standards

#### Excellent sensitivity using serum <sup>1</sup>

	Population	Determine® Syphilis TP	Test A	Test B
Specificity	Non-syphilis	100%	100%	100%
Sensitivity	Syphilis	100%	97.73%	94.32%

#### Highly accurate using whole blood and paired serum and plasma <sup>2</sup>

	Population	Determine® Syphilis TP			Test A Serum	Test B Serum
		Whole Blood	Plasma	Serum		
Specificity	Non-syphilis	100%	100%	100%	100%	100%
Sensitivity	Syphilis	92.31%	100%	100%	98.08%	94.23%

**1.** Serum specimens from 325 cases of non-syphilis and 176 cases of syphilis from three clinical sites in Japan were tested by Determine® Syphilis TP and two commercially available tests.

**2.** Whole blood specimens with paired serum and plasma from 47 cases of non-syphilis and 52 cases of syphilis from three clinical sites in Japan were tested by Determine® Syphilis TP and two commercially available tests.

[Determine HIV Test](#) | 
 [Determine HIV-1/2 Ag/Ab](#) | 
 [Determine Hepatitis B Test](#) | 
 [Determine Syphilis Test](#) | 
 [Site Map](#)

Determine HIV-1/2 Ag/ Ab Combo is currently not available in the US

