

- [HIV 1/2](#)
- [Features & Benefits](#)
- [Determine HIV 1/2 Performance Data](#)
- [Determine HIV 1/2 How to Use](#)
- [Determine HIV 1/2 Product Specifications](#)
- [Determine HIV 1/2 FAQ](#)



Performance Data

Determine HIV 1/2

Determine® HIV-1/2 delivers outstanding performance versus industry standards.

Excellent sensitivity ¹

Population	Determine® HIV-1/2	Test A
HIV-1 positive	100%	100%
HIV-2 positive	100%	100%
HIV-1 Subtypes A-G	100%	Not tested
HIV-1 Group O	100%	Not tested
Total	100%	100%

Excellent specificity ²

Population	Determine® HIV-1/2	Test A
Seronegatives - Serum	99.89%	100%
- Plasma	100%	100%
Pregnant females	100%	100%
West Africans	97.96%	97.96%
Disease states other than HIV and potentially interfering substances	98.86%	99.45%
Total	99.75%	99.87%

1. A total of 869 HIV-1 and HIV-2 antibody positive serum and plasma specimens from Asia, Africa, North America and South America were tested by Determine® HIV-1/2 and a commercially available test.

2. A total of 1,594 serum and plasma specimens from Asia, West Africa and North America were tested by Determine® HIV-1/2 and a commercially available test.

[Determine HIV Test](#) |
 [Determine HIV-1/2 Ag/Ab](#) |
 [Determine Hepatitis B Test](#) |
 [Determine Syphilis Test](#) |
 [Site Map](#)

Determine HIV-1/2 Ag/ Ab Combo is currently not available in the US